


**IMPORTANT: BEFORE CHANGING PAPER, MACHINE MUST BE POWERED ON.**

**TEAR OFF FIRST 6 INCHES OF PAPER TO PREVENT ADHESIVE FROM DAMAGING PAPER CUTTER.**

1. Make sure Machine is powered on.

Remove old paper and paper roller.

Lift up cutter assembly.

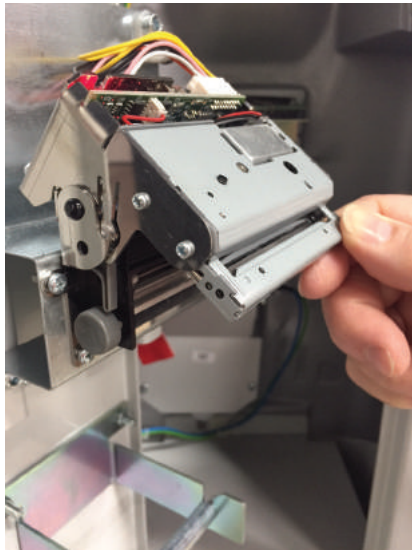

2. Place new paper roll in holder, with BioMeasure logo facing you.

**ALIGNMENT IS CRITICAL**

Tear paper straight across and insert behind black roller, using both hands as shown.



3. Rotate gray knob on left downward, if necessary, to advance paper until paper self-feeds and STOPS.


Push paper feed button to advance paper further, if necessary.

If paper is not aligned, this step will need to be repeated to obtain proper paper alignment.



4. Insert paper through cutter assembly slot as shown.


Pull end slightly to prevent folding.

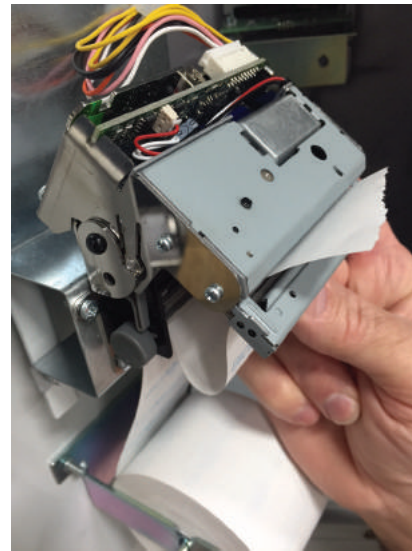

5. Hold end of paper to prevent folding.

Push down on cutter assembly

**NOTE:  
GRAY LATCH MUST BE DOWN BEFORE PUSHING DOWN CUTTER.**

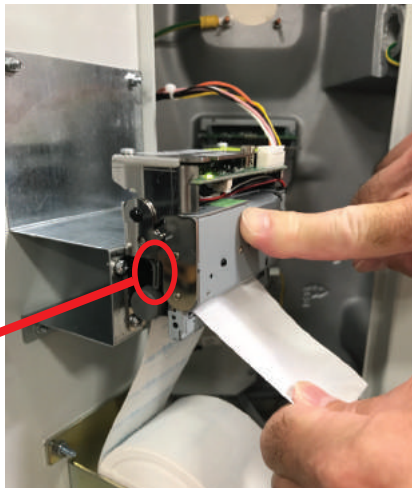

6. Push paper feed button for several seconds to confirm paper alignment.

Release paper feed button to confirm paper cuts.



Paper Feed Button